# MULTI ACTION 5

**Virucide - Bactéricide – Fongicide -Tuberculocide**

**DIN#02264021**

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### GUARANTIE:

|  |       |
|--|-------|
| Alkyl (C <sub>14</sub> 50%, C <sub>12</sub> 40%, C <sub>16</sub> 10%) Dimethyl Benzyl Ammonium Chloride..... | 0.20% |
| Octyl decyl Dimethyl Ammonium Chloride.....  | 0.15% |
| Dioctyl Dimethyl Ammonium Chloride.....  | 0.06% |
| Didecyl Dimethyl Ammonium Chloride.....  | 0.09% |

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# MULTI ACTION 5

DIN#02264021

READY TO USE WIDE SPECTRUM DISINFECTANT FOR MEDICAL EQUIPMENTS

VIRULICIDAL - BACTERICIDAL- FUNGICIDAL-TUBERCULOCIDE

|  |                 |
|--|-----------------|
| Didecyl Dimethyl Ammonium Chloride:  | 0.09%           |
| Alkyl (C <sub>14</sub> 50%, C <sub>12</sub> 40%, C <sub>16</sub> 10%) Dimethyl Benzyl Ammonium Chloride: | 0.2%            |
| Octyl decyl Dimethyl Ammonium Chloride:  | 0.15%           |
| Dioctyl Dimethyl Ammonium Chloride:  | 0.06%           |
| Isopropanol:   | 44.0%           |
| Other ingredients:   | Balance to 100% |

## DESCRIPTION

**MULTI ACTION 5** is a wide spectrum disinfectant that is efficient against : virus, bacteria, yeast, mold and algae. **MULTI ACTION 5** is approved by **Health Canada** as a wide spectrum disinfectant. **MULTI ACTION 5** belongs to a family of Isopropanol Poly-Quaternary compounds mixture that are environmentally friendly.

## DILUTON

Use **MULTI ACTION 5** as is. Vaporise **MULTI ACTION 5** to disinfect medical instruments and surfaces. The efficiency of **MULTI ACTION 5** exceeds all known disinfectants.

**MULTI ACTION 5** contains 5000 ppm of PolyQuaternary ammonium chloride. This ingredient:

- meets AOAC Use-Dilution Test Standards for hospital disinfectants.
- meets EPA's Sanitizer Test standards for non-food contact surface sanitizers. At 200 ppm active, is an effective one-step sanitizer by eliminating 99.9% of *Staphylococcus aureus* and *Klebsiella pneumoniae* in 60 seconds.

It is an effective sanitizer for use on food contact surfaces in 60 seconds at 200 ppm active and 500 ppm hard water against:

- *Campylobacter jejuni*
- *Escherichia coli*
- *Escherichia coli* 0157:H7
- *Klebsiella pneumoniae*
- *Listeria monocytogenes*
- *Salmonella choleraesuis*
- *Shigella sonnei*
- *Staphylococcus aureus*
- *Yersinia enterocolitica*

It is an effective sanitizer for use on food contact surfaces in 60 seconds at 400 ppm active and 1000 ppm hard water against:

- *Escherichia coli*
- *Staphylococcus aureus*
- It is effective against Citrus Canker Disease (*Xanthomonas axonopodis* pv. *citri*) at 2000-ppm active quat.
- It is effective against:
- Avian Infectious Bronchitis Virus
- Avian Influenza Virus
- Canine Distemper Virus
- Marek's Disease Virus
- Newcastle's Disease Virus
- Pseudorabies Virus
- Arkansas '99 (Infectious Bronchitis Virus)
  
- It is a Hospital Disinfectant at (450 ppm active), Bactericidal according to the current AOAC Disinfectants Use-Dilution Method, Fungicidal according to the AOAC Fungicidal Test, and Virucidal\* according to the virucidal qualification, modified in the presence of 5% organic serum against:

It kills (*Bacteria*):

- *Pseudomonas aeruginosa* (*Pseudomonas*)
- *Staphylococcus aureus* (*Staph*)
- *Salmonella choleraesuis* (*Salmonella*)
- *Brevibacterium ammoniagenes* (*Brevibacterium*)
- *Campylobacter jejuni* (*Campylobacter*)
- *Escherichia coli* (*E. coli*)
- *Escherichia coli* pathogenic strain 0157:H7 (pathogenic *E. coli*)
- *Klebsiella pneumoniae* (*Klebsiella*)
- *Listeria monocytogenes* (*Listeria*)
- *Pseudomonas cepacia* (*Pseudomonas*)
- *Salmonella typhi* (*Salmonella*)
- *Yersinia enterocolitica*

It kills (*Viruses*):

- \*Herpes Simplex Type 1 (*Herpes*)
- \*Herpes Simplex Type 2 (*Herpes*)
- \*Influenza A2 / Hong Kong (*Influenza*)
- \*Vaccinia (*Pox Virus*)

It kills (*Fungi*):

- *Trichophyton mentagrophytes* (*Athlete's Foot Fungus*)
- (*Cause of Ringworm*)